# How to use your new Rehband Support - 7725

RU

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UK

## Function

Top with integrated heat zones on the shoulders, neck, lumbar and pelvic region in combination with the flexible material Cool Max Lycra.

## Indication

Preventive effect, improved muscle co-ordination (proprioception effect) Alleviates aching muscles.

## Material

1,5 mm SBR/neoprene, polyamide jersey, Lycra

## Important Information

- 1) If you have questions regarding application, please contact your local dealer or the prescriber.
- 2) The product is not flameproof, avoid temperature above 120°C/ 248°F.
- 3) Minor changes are allowed, providing the changes are made through the prescriber\* or local dealer.
- 4) The function of the product must not be changed or compromised, the product should only be used for its original, intended indication.
- 5) Re: SBR/neoprene products, time of use should not exceed 3-4 hours. Re: products in other materials, time of use should not exceed 8 hours, unless otherwise is recommended by prescriber.
- 6) If the product contains latex or rubber and you are allergic to latex/ rubber, we advise you to consult a prescriber before using the support. The content of the product is written above.
- 7) If you are allergic to heat, we advise you to use a product that not contains SBR/neoprene.

\*Re: Medical- products, the prescriber must be a qualified doctor, PT, OT or Orthopaedic Engineer.

## Washing Instructions

- 1) Close all Velcro.
- 2) Remove detachable splints.
- 3) Mesh laundry bag is recommended.
- 4) Stretch product while wet and let dry.

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